

REICHERT PRODUCT CLEANING & DISINFECTION INSTRUCTIONS

DESCRIPTION

Reichert remains committed to the safety and well-being of all eye care professionals using our devices and protecting and promoting the health of all patients that benefit from them every day. This includes the proper cleaning and disinfection of our devices, which is a critical aspect of patient care and office safety.

Details

Reichert has been extensively evaluating the cleaning and disinfection agents for all our devices due to current guidelines. This cleaning and disinfection information is available on our website at: www.reichert.com/disinfection

Our web page provides a Cleaning and Disinfection Matrix document with the most up-to-date approved list and will be updated as additional information becomes available. Please check this webpage for the latest updated information.

NOTE: Although the web page will allow the PDF document to be printed, please be sure to review the web page for any current updates after the document has been printed.

SAMPLE MATRIX:

Below is a sample of the Cleaning and Disinfection Matrix from our web page. Please visit our web page for the **current version** of the matrix: www.reichert.com/disinfection

CLEANING AND DISINFECTION MATRIX FOR REICHERT® DEVICES								
The following chart lists the approved cleaning solutions that can be used on Reichert® devices. Follow the guidelines below when cleaning and disinfecting your device. Do not use cleaning solutions that are not approved by Reichert or damage to your device may occur. For further information please contact Reichert Technical Support at 1-888-849-8955 (toll-free in US and Canada) or reichert.support@ametek.com .								
CLEANING MATRIX								
Reichert Product and Model Number	Specific Parts and Accessories	CLEANING SOLUTIONS						Notes
		Soap and Water*	70% Isopropyl Alcohol	5.25% Bleach and Water Solution	7.5% Hydrogen Peroxide and Water Solution	Sani-cloth Prime Germicidal Wipes**	Ethyl Alcohol	
Phoropter® VRx 16241/16242	Body	✓	✓	✓	✓	✓	✗	
	Face Shields	✓	✓	✓	✓	✓	✗	
	Headrest	✓	✓	✓	✓	✓	✗	
	Screen	✓	✓	✓	✓	✓	✗	
	Controller Buttons	✓	✓	✓	✓	✓	✗	
Visutron 900 Touch 16243/16244/ 16245/16246/ 16247/16248	Body	✓	✓	✓	✓	✓	✗	
	Face Shields	✓	✓	✗	✗	✗	✗	
	Headrest	✓	✓	✗	✗	✗	✗	
	Screen	✓	✓	✓	✓	✓	✗	
	Controller Buttons	✓	✓	✓	✓	✓	✗	
Ultramatic RX Master™ Phoropter® 11625B/11625W 11635B/11635W	Body	✓	✗	✓	✓	TBD	✗	Do not use lint generating materials.
	Face Shield	✓	✓	✓	✓	TBD	✓	
	Forehead Rest	✓	✓	✓	✓	TBD	✓	
	Lenses	TBD	TBD	TBD	TBD	TBD	TBD	Use a rubber ear syringe to blow dust, a softwood stick with cloth and lens cleaner, and/or 'camel's hair brush'.



© 2020 AMETEK, Inc. A Reichert, Inc. ISO 13485 Certified - Reichert products are designed and manufactured under quality processes meeting ISO 13485 requirements.

July 15, 2020 Rev. A