

# Instructions for cleaning and sterilization

Lensmeter  
VX40

# VISIONIX

This instruction guide aims to clean and sterilize your VX40 for common use .  
For other instructions, please refer to the USER MANUAL.

For your comfort it's recommended to clean the device regularly. Please clean the instrument following these instructions. Cleaning conducted by a process other than the one described in this instruction may cause deterioration or damage to the equipment.



## **1- Cleaning of frame or lenses**

Cleaning of the frame or lens before analysis is important.  
Clean the frame with idalkan wipes incl. alcohol. disinfectant or other alternative.

## **2- Cleaning of lens clamping**

Cleaning of the lens clamping important.  
Clean the frame with Cidalkan wipes incl. alcohol. disinfectant or other alternative.  
Pay particular attention



During the lens clamping cleaning do not clean do stongly you may damage this part..

### **3- Cleaning of screen**

The screen should not be seen or touched by the patient. But clean the screen regularly with Cidalkan wipes or other alternative. Recommendation is to clean it between each patient, after you take off your gloves.



The screen must be wiped gently without applying excessive force.

### **4- Cleaning of on/off button**

The button should be touched only by the professional but clean the button regularly with Cidalkan wipes or other alternative. Recommendation is to clean it between each patient, after you take off your gloves.

### **5- Cleaning of the external body of the device**

Proceed to a regular cleaning of the exterior of the device.

Before cleaning the unit, turn it off and unplug it from the electric outlet.

To clean the plastic surface of the device gently wipe the top, bottom, and front surfaces.



Only use the below recommended solutions. Other solutions may damage or discolour the device.



Avoid to give the printed results to your patients.




## **Wipes - Instructions for use please read carefully !**

01




Take a wipe out of its packaging and close the packaging tightly after use.

02

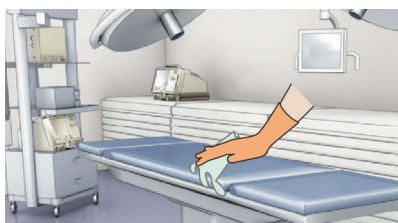

Clean thoroughly by hand wiping. Leave the product on for 5 minutes. Do not rinse.

03

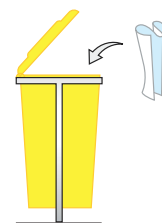

Throw away the wipe. Change wipe for each medical surface or device.

REF. 369113 Cidalkan wipes incl. alcohol.  
Standards CE 0459



**PRODUCT  
DISINFECTED**



**Luneau Technology**  
**LUNEAU TECHNOLOGY OPERATIONS**

2 Rue Roger Bonnet-27340 PONT DE L'ARCHE - FRANCE - Tel. + 33 232 989 132 • [contact@luneautech.com](mailto:contact@luneautech.com) • [www.luneautech.com](http://www.luneautech.com)